

Pet Piller Is The Perfect Tool To Save Your Fingers When Administering Tablets And Stops Your Pet From Chewing Or Spitting Out Medication

The new improved design makes it easy to use and gives more control when administering tablets and capsules to your cat or dog. If you struggle to give your pet their regular worming tablet or medication, then this product is for you!

- This easy-to-use device makes dosing your pets much faster, easier and safer for you and your pet
- Great to prevent your fingers from getting bitten when trying to administer with your hands
- Allows you to deliver the medication at the back of your pet's throat and stops your pet from chewing or spitting the tablet out
- Includes finger grips for easy application
- Soft and gentle blue rubber tip firmly holds the pill in place
- Easy dispensing mechanism for fast dosing
- Gentle for your pet's mouth
- Recommended by veterinarians
- Ideal for cats and dogs



HOW TO USE

1. Place the tablet in the prongs of the blue rubber end of the device
2. Place fingers on finger grooves on opposite end
3. Open your pet's mouth using your free hand
4. Push the round trigger at opposite end of piller device to dispense the pill at the back of your pet's throat
5. Hold your pet's mouth closed and tickle under their chin to make sure they have swallowed the pill

