



# Finger Brushes

Created by Global Veterinary Products  
Distributed by [www.prestigepetproducts.com.au](http://www.prestigepetproducts.com.au)

**The GVP Finger Brush is an easy to use product for brushing your pets teeth. Simply slip the bristled tip over your finger and brush away! By moving your finger along the surface of your pets teeth, you wont have to maneuver awkward, pokey or sizeable toothbrushes.**

Regular toothbrushing is one of the most effective ways to reduce dental plaque and tartar build up for better oral health. Learn more about preventing dental disease in dogs.

Simply slip over your finger and start brushing  
No need to coordinate yourself with a bulky brush - just move your finger!  
Ergonomically designed brush is also dishwasher safe



## INTRODUCING YOUR PET TO TEETH BRUSHING

- Cats and small dogs may feel more comfortable if they can sit on their owners lap while having their teeth brushed.
- Begin slowly, initial sessions should be brief, a minute or two and well rewarded.
- Get your pet used to the toothbrush by dipping it in tuna juice, chicken or beef stock or just use water.
- Next try offering the toothbrush with the paste, without brushing. Allow your pet to taste the paste.
- When your pet is comfortable with the brush try brushing one or two strokes on a few teeth. Slowly increase the amount of brushing as your pet becomes more comfortable.
- Start at the front of the mouth. Pets are often more accepting of this.